

COVID-19 Sample Collection Kit



※ What is COVID-19 Sample Collection Kit?

This kit is called UTM(Universal Transport Medium). UTM is intended for the collection and transport of clinical specimens containing Corona Virus and others.

※ How to use it?



Fig.1



Fig.2

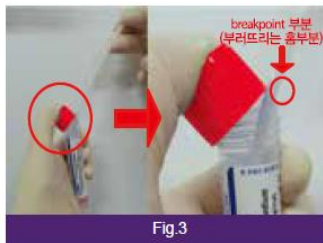


Fig.3



Fig.4

[Fig.1] Peel open the sealed pouch pack.

[Fig.2] Remove one swab from the pouch and collect the specimen without bending the swab.


[Fig.3] Aseptically remove the cap from the vial and insert the swab into the tube containing medium.

Break the swab shaft by bending the swab.

[Fig.4] Replace the cap and secure the lid tightly and record the patient's information on the label.

※ How to be made up?

Cat.No	Description
UTNFS-3B-2	Viral Sample Collection Kit ((Nasopharyngeal swab) + (Oropharyngeal swab) + 2ml buffer)
	PACKAGE : 50EA/1 innerbox, 10Innerbox/1 cartonbox

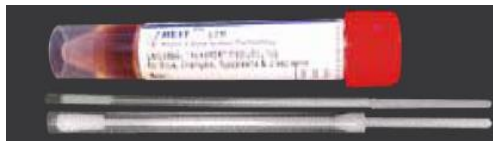

※ Why 2swabs?

For initial diagnostic testing for COVID-19, CDC(Centers for Disease Control and prevention) recommends collecting and testing upper respiratory (nasopharyngeal AND oropharyngeal swabs)